




Ukast Elite POP bandage consist of a high quality cotton gauze coated with formulated plaster of paris powder. It is designed for rapid and even wetting upon immersion. It forms a hard and rigid cast within 2 - 3 minutes of setting time while early weight bearing is achieved in 30 minutes.





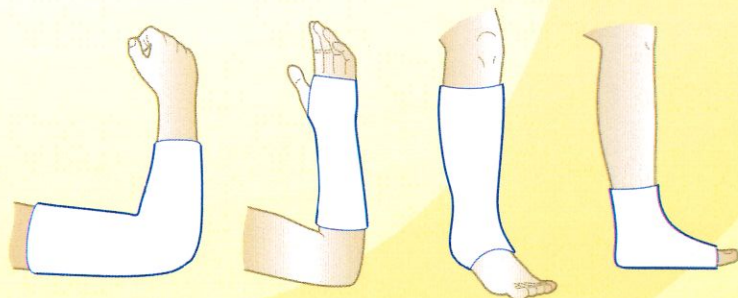

An immobiliser for acute and secondary fractures, sprains and for corrective procedures of other body parts normally affecting upper and lower limbs. It is applied as a full cylindrical cast and/or back slab/ splint.

Features

- High strength
- Creamy structure
- Low plaster loss
- Excellent moulding
- Uniform and fast setting



Operating Instructions

		
<p>1 Put a plaster liner on the affected area.</p>	<p>2 The doctor put on his gloves.</p>	<p>3 Open the package and remove the bandage.</p>
		
<p>4 Soak the bandage in water for about 5 seconds.</p>	<p>5 Wrap the bandage around the affected area.</p>	<p>6 Smooth and wait for the bandage to solidify.</p>

Product variant

Size	Identifier	Packing	Size	Identifier	Packing
5*270cm	EP5	192 rolls/carton	12.5*270cm	EP12.5	72 rolls/carton
7.5*270cm	EP7.5	144 rolls/carton	15*270cm	EP15	72 rolls/carton
10*270cm	EP10	96 rolls/carton	20*270cm	EP20	48 rolls/carton

Manufactured by:

Shanghai YouJia Medical Supply Co., Ltd.

No. 499, Fengcun Road, Fengxian District, Shanghai, China

Tel: +86 021 5756 9919 / +86 021 5756 9920 | Mob: +86 186 2111 8038 | Email: info@ukast.cn

A.R Importer:

NAEL Marketing (Co.Reg: 002039651-A) (MDA Product Registration: GA3631720-44882)

Level 23-1, Premier Suite One Mont Kiara, No.1, Jalan Kiara Mont Kiara, 50480 Kuala Lumpur, Malaysia.

Tel: +603 2785 6818 | Mob: +6012 232 9077 | Email: nmktg21@gmail.com



MDA reg no. GA4964020-47413